



IMPACTO® FOOT PROTECTION

Viscolas Anti-Fatigue Heel SPUR Cushions

- > Viscolas® is the only visco-elastic polymer formulated for orthopedic application
- > Help reduce heel, foot, knee, leg and low back pain
- > Designed to attenuate skeletal shock, reducing potentially damaging forces on weight-bearing joints
- > Non-irritating, resistant to fungal and bacterial growth and can easily be cleaned with soap and water
- > Great for arthritis of weight-bearing joints and postoperative conditions
- > Used to provide relief from fatigue and pain or to help with conditions such as tendonitis, shin splints, plantar fasciitis or heel spur
- > Can be comfortably worn with low-cut dress shoes, sports shoes, work shoes and boots
- > Soft viscoelastic polymer won't "bottom out" with special surface tackiness adhesive to keep in place, moisture wicking fabric cover
- > Natural tackiness of components can be increased with an application of alcohol or decreased with a dusting of talcum powder
- > Made in USA, USDA Accepted



Anti-Fatigue Heel Relief Spur Cushion

Provide maximum shock protection and pressure reduction for patients suffering from calcaneal heel spurs or other acutely painful heel conditions. A removable plug allows for a cavity to be created beneath the area of greatest discomfort, while extra cushioning on the medial side of the plantar fascia provides added relief.

PART No.	USA Men	USA Women	UK	EUROPE
VI4838A	2 - 4	4 - 6	1 - 3	33 - 36
VI4839B	5 - 7	7 - 9	4 - 6	37 - 40
VI4840C	8 - 10	10 - 12	7 - 9	41 - 44
VI4841D	11 - 13	12 - 13	11-12	45 - 48

Place heel spur cushion into the heel area of the shoe with the teardrop pointing towards the inside of the heel.
 Hand wash or wipe the surface with rubbing alcohol or warm soapy water. Hang to dry. Do not dry clean or place in dryer.
 The natural tackiness of the components can be increased with an application of alcohol or decreased with a dusting of talcum powder.



Sold per pair. 50 per case.



TRUSTED PROTECTION

ISO 9001:2015



IMPACTO.CA

N. America 888 232 0031
 Free phone UK 0800 0280 243
 T: 613 966 0062 F: 613 966 0067
 PO BOX 524, BELLEVILLE, ON K8N 5B2
impacto@impacto.ca

Distributed by.