

Manufacturer's Instructions and User Information

Name and Address of the manufacturer: Impacto Protective Products Inc.
PO Box 524, Belleville, ON K8N 5B2 Canada

Style Identification: Air Temp Advantage

Description: **IMPACTO® Air Temp Advance Back Support.** Built in pockets rotate off the back to insert air support chambers (included), cold compression or moist heat (optional). Neoprene inner lining. Plastic support stays with non-slip strips. Two air support chambers with self-contained aspirator when inflated conform to the back providing a natural massaging action that reduces muscular back discomfort. Elastic side straps attached with hook and loop. Adjustable hook and loop closure.



These garments bear CE marking to demonstrate compliance with EU 2016/425 Personal Protective Equipment Annex II Health & Safety Requirements.
EU Declaration of Conformity: <https://www.impacto.ca/products/datasheet-ce-conformity-statements/>

Fitting Instructions:

Inflate air chamber by squeezing hole at bottom to desired stiff ness. Fold bottom up and slide into pocket with folded side first. Hook fastener to secure pocket in center position. If unhooked, pocket will move to side position when fastening straps. If over time bladder softens, squeeze hole to re-supply air. Loosen elastic straps. Position belt just below the navel. Attach front closure to desired level of comfort. Before you lift, or when additional support is desired, grasp ends of elastic straps, pull around to the front and attach hook closures. Secure elastic straps so that you are comfortable. When lifting is completed, loosen straps and re-attach with less tension to some point further back on your side. Suspenders may be attached at the waist with the hook and loop fasteners provided.

Size	Waist Measurement		
Small	25 – 28 inch	61 – 81 cm	
Medium	29 – 35 inch	82 – 94 cm	For sizing, use waist measurement or add 2" if belt is worn over clothing.
Large	36 – 41 inch	91 – 107 cm	
X-Large	42 – 50 inch	104 – 122 cm	
XX-Large	51 – 52 inch	117 – 132 cm	
XXX-Large	57 – 62 inch	134 – 147 cm	

WEIGHT: 0.90 LB, SOLD PER EACH. CASE: 10.

Washing Instructions:

Hand wash in cold water and mild detergent. Do not use bleach or strong detergent. Hang or lay flat, air dry. Do not dry in automatic dryer.

Limitations of Use:

These products are physiological aids. It is important to wear the correct size and comfortably fastened at all times. This is only a support belt and should not be considered a remedy for back problems. If you are under the care of a health practitioner, consult with them before using your Back Coach. This belt will not guarantee freedom from all back strain. Always utilize good judgment and proper lifting techniques, especially when objects are below 30" from the floor.

After Care:

Follow the manufacturer's recommendations for washing and cleaning. To ensure optimum protection, regular comparisons are recommended against a new product. There are no other warranties expressed or implied. The company will not accept liability for product, whose use and care advice has been ignored.

EU DECLARATION OF CONFORMITY

We,

Impacto Protective Products Inc.
PO Box 524
Belleville, ON K8N 5B2
Canada

Declare that the PPE described here after,

ATA (Air Temp Advantage)

These garments bear CE marking to demonstrate compliance with EU
2016/425 Personal Protective Equipment Annex II Health & Safety Requirements.

this declaration of conformity is self-certified and issued under the sole responsibility of the
manufacturer;

The technical Construction File is maintained at the following addresses:

1. Impacto Protective Products Inc.
PO Box 524
Belleville, ON K8N 5B2 Canada
Tel 0613 9660 062
Fax 0613 9660 067
UK free phone 0800 0280 243
Email impacto@impacto.ca
Contact Mr. Eric Lehtinen

Authorized Signatory



Date Sep 9, 24

Name:

Eric Lehtinen

Position:

President
Impacto Protective Products Inc.