

## Manufacturer's Instructions and User Information

**Name and Address of the manufacturer:** Impacto Protective Products Inc.  
PO Box 524, Belleville, ON K8N 5B2 Canada

**Style Identification:** Relievers®

**Description:** **IMPACTO® Anti-Fatigue Reliever Insoles**

Flat footbeds, 1/8" thick. Made of a soft viscoelastic material that won't bottom out under the weight of your body or with lots of wear, these insoles will cushion the shock that is created at heel strike. Fabric cover wicks away moisture. A special surface tackiness adhesive keeps these foot pain relief insoles in place to prevent bunching at the toe area.



These garments bear CE marking to demonstrate compliance with EU 2016/425 Personal Protective Equipment Annex II Health & Safety Requirements.

EU Declaration of Conformity: <https://www.impacto.ca/products/datasheet-ce-conformity-statements/>

**Fitting Instructions:**

If your shoe size does not correspond exactly to the size range indicated, you may trim the front part of the insole with a pair of scissors. **DO NOT TRIM THE HEEL AREA.** Before inserting the insole remove with care any existing insoles from the shoe. Insert the new insole into the shoe. Further trimming may be necessary to obtain a perfect fit.

Part#	USA Men	USA women	UK	Europe
VI4781A	2 – 4	4 – 6	1 – 3	33 - 36
VI4782B	5 – 7	7 - 9	4 – 6	37 – 40
VI4783C	8 – 10	10 – 13	7 – 9	41 – 44
VI4784D	11 - 13		10 - 12	45 - 48
VI4785D	14 - 16		13 - 14	48 - 50

Placed in shoes with sticky side down. Insoles may be used with taping, orthotics or any other components of the Orthotic Component System.

Weight (Lbs.): 0.40, Sold per Pair. Case: 50.

**Washing Instructions:**

Hand wash or wipe the surface with rubbing alcohol or warm soapy water. Hang to dry. Do not dry clean or place in dryer. The natural tackiness of the components can be increased with an application of alcohol or decreased with a dusting of talcum powder.

**Limitations of Use:**

It is recommended that, once inserted, these cushions remain in the shoes. Continual removal from one pair of shoes to another may cause deterioration.

**After Care:**

Follow the manufacturer's recommendations for washing and cleaning. To ensure optimum protection, regular comparisons are recommended against a new product. This product is warranted against defects in materials and workmanship for 12 months from purchase. There are no other warranties expressed or implied. The company will not accept liability for product, whose use and care advice has been ignored.

# EU DECLARATION OF CONFORMITY

We,

Impacto Protective Products Inc.  
PO Box 524  
Belleville, ON K8N 5B2  
Canada

Declare that the PPE described here after,

Blue dot, Heel Spur, Heel Cups, Reliever Insoles

These garments bear CE marking to demonstrate compliance with EU  
2016/425 Personal Protective Equipment Annex II Health & Safety Requirements.

this declaration of conformity is self-certified and issued under the sole responsibility of the  
manufacturer;

The technical Construction File is maintained at the following addresses:

1. Impacto Protective Products Inc.  
PO Box 524  
Belleville, ON K8N 5B2 Canada  
Tel 0613 9660 062  
Fax 0613 9660 067  
UK free phone 0800 0280 243  
Email [impacto@impacto.ca](mailto:impacto@impacto.ca)  
Contact Mr. Eric Lehtinen

Authorized Signatory



Date Oct 24, 24

Name:

Eric Lehtinen

Position:

President  
Impacto Protective Products Inc.