Manufacturer's Instructions and User Information

Name and Address of the manufacturer: Impacto Protective Products Inc.

PO Box 524, Belleville, ON K8N 5B2 Canada

Style Identification: UPGUARD3000

Description: IMPACTO® Neck Support System. Helps to prevent over extension

reducing neck pain, fatigue & injuries. Ideal for workers that must look up for sustained periods. Made with molded, soft foam. Adjustable webbing straps

with large capacity clips to fasten at waist One size.





These garments bear CE marking to demonstrate compliance with EU 2016/425 Personal Protective Equipment Annex II Health & Safety Requirements.

EU Declaration of Conformity: https://www.impacto.ca/products/datasheet-ce-conformity-statements/

Fitting Instructions:

Place the UPGUARD on your shoulders over the upper/top of back and clip the two fasteners to either your belt or pant tops. The adjustable straps should be positioned to provide a loose fit that does not pull on the shoulders. Moving the clips further out from the center of the stomach area will provide better stability.

Sizing: One size.

Weight (Lbs.): 0.50. Sold per Each. Case: 20.

Washing Instructions:

Hand wash in cold water and mild detergent. Air dry. Do not machine wash or place in dryer. Allow webbing to completely dry before re-using.

Limitations of Use:

These products are physiological aids. It is important to wear it comfortably fastened at all times. Do not wear the UPGUARD if you suffer from severe Rheumatoid Arthritis, Ankylosing Spondylitis, Reiter's Syndrome, Psoriatic Arthritis, Downs Syndrome, Enteropathic Arthritis or any condition predisposing upper neck instability. If there is any question consult physician prior to use.

If the potential of flame or heat exists, the use of the disposable UPGUARD COVER, sold as accessory, is recommended. This cover should not be laundered and should be discarded after each use.

After Care:

Follow the manufacturer's recommendations for wear and cleaning. To ensure optimum protection, regular comparisons are recommended against a new product. The company will not accept liability for product, whose use and care advice has been ignored.

EU DECLARATION OF CONFORMITY

We,

Impacto Protective Products Inc.

PO Box 524

Belleville, ON K8N 5B2

Canada

Declare that the PPE described here after,

UPGUARD3000

These garments bear CE marking to demonstrate compliance with EU 2016/425 Personal Protective Equipment Annex II Health & Safety Requirements.

this declaration of conformity is self-certified and issued under the sole responsibility of the manufacturer;

The technical Construction File is maintained at the following addresses:

1. Impacto Protective Products Inc.

PO Box 524

Belleville, ON K8N 5B2 Canada Tel 0613 9660 062 Fax 0613 9660 067 UK free phone 0800 0280 243

Email impacto@impacto.ca
Contact Mr. Eric Lehtinen

Authorized Signatory

Date Sep 9, 24

Name:

Eric Lehtinen

Position:

President

Impacto Protective Products Inc.